Udskiftning aktuator / Changing the actuator / Utskifting/byte av motor seteløft - VELA Tango El

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ENGLISH

1. Use the special tool (A) to remove the frame. You can see how to use the tool by using the QR code above.
2. Pull off the cover (B) under the seat (fastened with velcro), unscrew the battery covers (C), and mark the wires and batteries (D) before removing the wires and removing the batteries. Unscrew the four screws (E) located on both sides of the actuator (F).
3. Mount the new actuator with the four screws (E) on both sides - tighten the screws well. Place the batteries (D) and reconnect the wires. **Important:** +/- poles must be correctly oriented. The motor control is damaged if it is installed inverted. Fasten the battery covers (C), and put the covers (B) in place again. Test that the actuator works by activating the height setting.
4. Place the plastic bushing (G) in the track on the actuator (F) at the desired height so that it does not slip out of the track. **Important:** Use only the top four tracks.
5. Place a knee on the seat to push the top piece fully in place in the frame. The bushing should be at level with the upper edge of the frame.

The special tool MUST be used because the electric lift cannot withstand shocks or bumps.