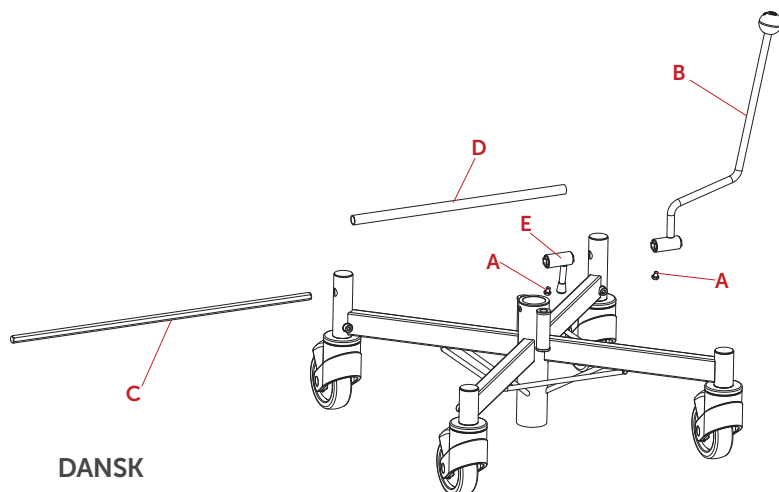
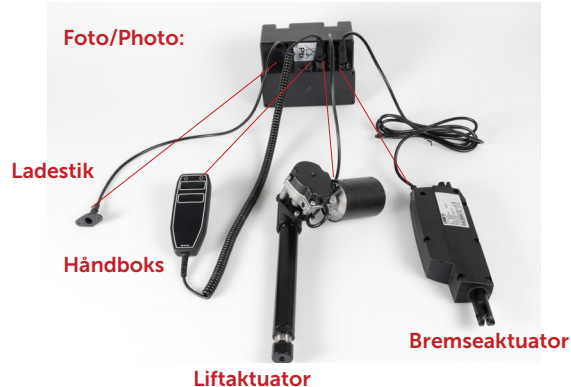
Montering af el-bremse/ Monterung av el-brems/ Montering av elbroms/ Mounting the electric brake - VELA Tango 300E/310E

Montering af el-bremse VELA Tango 300E/310E
Varenummer/ Artikelnummer/ Product number:
(A)928957 EL-bremsekit Tango 300E/310E
m/motorstyring, aktuator, kontaktboks

Foto/Photo:

**DANSK***Afmontering af bremseaksel*

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ENGLISH

Removing the brake shaft

1. Unscrew the 2 bolts (A) and remove the brake rod (B). **2.** Knock out the hexagon shaft (C) completely and remove the protecting tube (D) and locking bracket (E). **3.** All these parts (A-E) are discarded. The frame is now ready for mounting the electric brake.

Mounting the electric brake

4. Place the protecting tube (F) next to the holes in the wheel towers (G) and push / knock the hexagon shaft (H) through the wheel towers and protecting tubes. The hexagon shaft must be positioned so that it protrudes equally from both wheel towers. **5.** Protecting tube and hexagon shaft is tightened with pin screw (I) – use Allen key. **Note:** Bracket (J) on protecting tube must face the frame. **6.** Place the locking ring (K) on the center tube (L) – the ring should be flush with the bottom of the center tube. **7.** Tighten the locking ring securely with the M10 screw (M) **8.** Place the brake actuator (N) (actuator must turn as shown) on the locking ring thread (O). tighten nut (P). **9.** Place the actuator (N) on the bracket on the protecting tube (F) and fasten the actuator with the M10 screw (Q) and lock nut (R).

Mounting of holder for hand box

10. Unscrew the middle screw (S) in the armrest cushion (T). **11.** Position the holder (U) and fasten it securely with the new screw and washer (V). **12.** Place the hand box in the holder.

Replacing motor control

13. Remove the 4 screws with washers (X) from the old motor control (Y). **14.** Position the new motor control (Z) and tighten the screws and washers (X) again. **15.** Insert the connectors into the ports as shown (see photo). Be sure to position the connectors correctly. **16.** Attach wires with strips to the chair to avoid wear and pinching.