Sejlsæde / Seilsete / hängsits / Sling seat - VELA Meywalk 4

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ENGLISH

1. Brake the gait trainer.
2. Remove the hand screw in the bottom of the centerbox on the gait trainer. Press in the hand screw and turn until the hand screw can be removed (A)
3. Remove the seat hanger from the centerbox by pulling it out from below (B)
4. Now mount the two arms for the sling seat on the inner side of the frame of the gait trainer (C,D)
5. Separate the brackets on the backside of the arms for the sling seat, by loosening the screws (E)
6. Mount the two arms for the sling seat on the inner side of the frame of the gait trainer by locking the brackets around the frame and tightening the screws (F)
7. Open the buckles on the sling seat and pull out the strap a little (G)
8. Mount the strap on the upper frame of the gait trainer by locking the buckle around the frame. Mount one strap on each side of the stanchion bar (H,I)
9. Mount the two other straps of the sling seat by putting the loop around the arms mounted on the frame of the gait trainer (J,K)
10. Tighten up the sling seat by pulling the two straps under the sling seat (K)