# Træk- og skubbestang / Trekk- og skyvestang / Tryck- och dragstång / Push and pull bar - VELA Meywalk 4

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### ENGLISH

#### Assembling the push and pull bar

1. Click the two bars together (A, B) in the length you want.
2. Loosen the screws in the bracket with the handle and mount it on the upper parts of the bar with the beak by tightening the screws (C,D)
3. Brake the gait trainer

#### Assembling the pull bar

4. Loosen the screws in the bracket without the handle and mount it on the lower frame on the front part of the gait trainer by tightening the screws (E)
5. Turn the bracket so the hole points upwards, tighten the screw in the hole until the bracket is locked to the frame of the gait trainer (F)

6. Grab the push and pull bar, open the beak in the bottom of the bar by pressing the ends together like a clothespin and lock the beak in the bracket on the gait trainer (G)

7. Press in the beak until it locks to the opposite end of the bracket (H)

#### Assembling the push bar

8. Dismount the rear stop of the gait trainer (I) by removing the screw in the bracket with an Allen Key
9. Insert the enclosed fitting in the cylinder, where the rear stop was placed and tighten the screw in the bracket with an Allen key (J,K)
10. Grab the push and pull bar, open the beak in the bottom of the bar by pressing the ends together like a clothespin and lock the beak in the bracket on the gait trainer. Press in the beak until it locks to the opposite end of the bracket (L)