Antitipstøtter / Anti-tip supports / Anti-tippstöd / Antikippstützen

DANSK

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
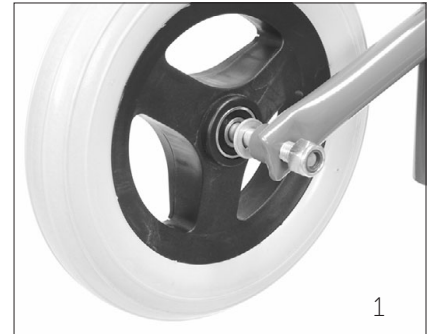
Antitipstøtter / Anti-tip supports / Anti-tippstöd / Antikippstützen

ENGLISH

Anti-tip supports are mounted on each side by first removing the centercap on the outside of the rear wheel and unscrewing the nut from the bolt which pass through the wheel hub (see figure 1). Two 19 mm (3/4") spanners will be needed for this. The longer bolt supplied with the anti-tip supports is fitted in the wheel hub instead of the existing bolt. Keep the old bolt safe in case the anti-tip supports later are removed. Then slide the U-fitting on the anti-tip support over the end of the frame where the wheel was mounted. Push the bolt through while placing the washer between the wheel and the U-fitting (see figure 2). Screw the nut back on. Remember to tighten firmly. Finally put on the center cap again.

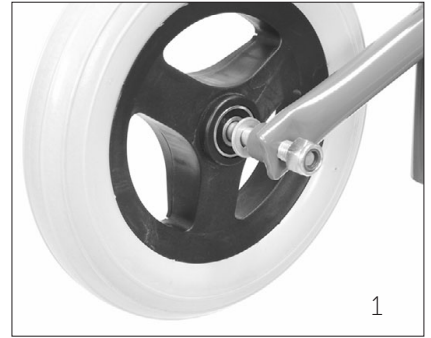The height of the anti-tip supports from the ground is adjusted by slackening the counter nut on the adjusting screw (see figure 2). This will require a 13 mm (1/2") spanner. If necessary slacken the nut on the wheel bolt. The anti-tip support is tilted up so the head of the adjusting screw rests against the end of the frame and the adjusting screw is screwed forward or back to get the correct height. Then retighten the counter nut and the nut on the wheel bolt.

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