



Overtræk til kropsring / Cover for trunk support


DANSK

Overtrækket til kropsringen på VELA Miniwalk dækker den forreste halvdel af ringen og gør den meget slidstærk.

For at sætte overtrækket på kropsringen må ringen tages af stellet. Dette gøres ved at løsne de to unbraco skruer på undersiden af kropsringen i hver side med en 6 mm unbracønøgle. Ringen kan nu let løftes af.

Overtrækket sættes på ved forsigtigt trække det ind over ringen i hver af de to ender - her må man naturligvis sikre sig at lynlåsen i overtrækket er åben. Lynlåsen skal sidde på undersiden af ringen og de to åbninger ved siden af lynlåsen skal passe med kropsringens skruehuller.

Når overtrækket sidder korrekt på ringen skrues ringen på stellet til VELA Miniwalk - gangvognen er igen klar til brug.

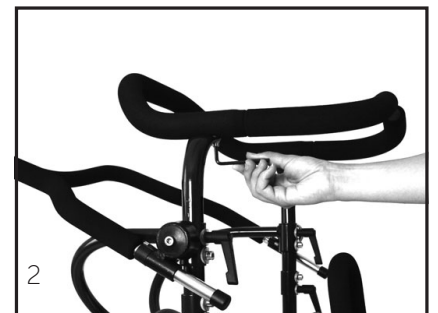



Overtræk til kropsring / Cover for trunk support

ENGLISH

The cover for the trunk support of VELA Miniwalk coats the front half of the trunk support and increases its resistance to wear.

In order to put the cover on the trunk support it is necessary to unscrew and remove the trunk support from the frame of VELA Miniwalk. This is done by loosening the two Allen screws at the underside of the trunk support on both sides using a 6 mm Allen key. The trunk support can now easily be lifted off.

Before putting the cover on the trunk support make sure the zip is open. Carefully pull the cover onto the trunk support while checking that the zip is placed at the underside. When the two openings fit the holes for the screws in the trunk support the cover is placed correctly.

Screw the trunk support back onto the frame of VELA Miniwalk and it is ready for use.

