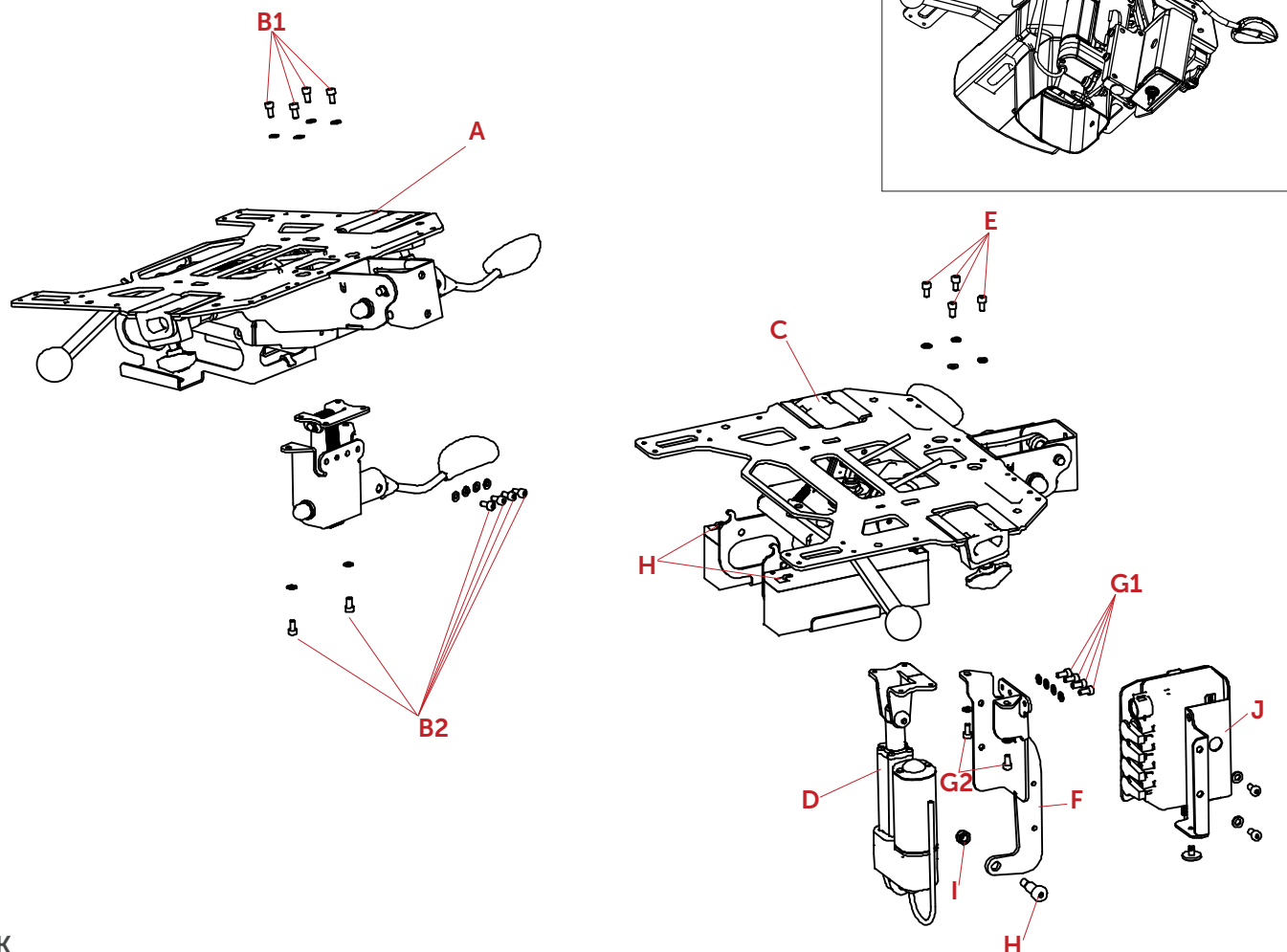
Elektrisk kip/ Elektrisk tilt/ Electrical tilt - VELA Tango 700

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ENGLISH

It is recommended that a VELA technician performs this assembly.

1. Remove the seat and battery covers (see instructions 106231 + 106235).
2. Tilt the tilt plate (A) forward to gain more space at the back.
3. Remove the existing seat tilt by removing the 10 screws and washers (B).
4. Install the top bracket (C) together with the actuator (D) as shown with 4 screws and washers (E) from above. The top bracket (C) needs to be tilted slightly to fit it into place against the tilt plate (A).
5. Install the large part of the bracket (F) on the bearing plate with the remaining 6 screws and washers (G).
6. Insert the lower pivot bolt (H) into the actuator (D) and bracket (F) and tighten the locknut (I).
7. Secure the motor controller (J) to the large part of the bracket (F) and tighten the wires.
8. Connect the power to the battery's positive and negative terminals (K) at the front of the bearing plate as shown. **Important:** +/- poles must be correctly oriented. If mounted incorrectly, the motor controller will be damaged.
9. Reinstall the seat and battery covers (see instructions 106231 + 106235). Tilt the tilt plate (A) completely forward to gain more space to fit the rear o-ring on the upper covers. Attach one o-ring to the rear of the two lower covers before mounting them onto the two upper covers.