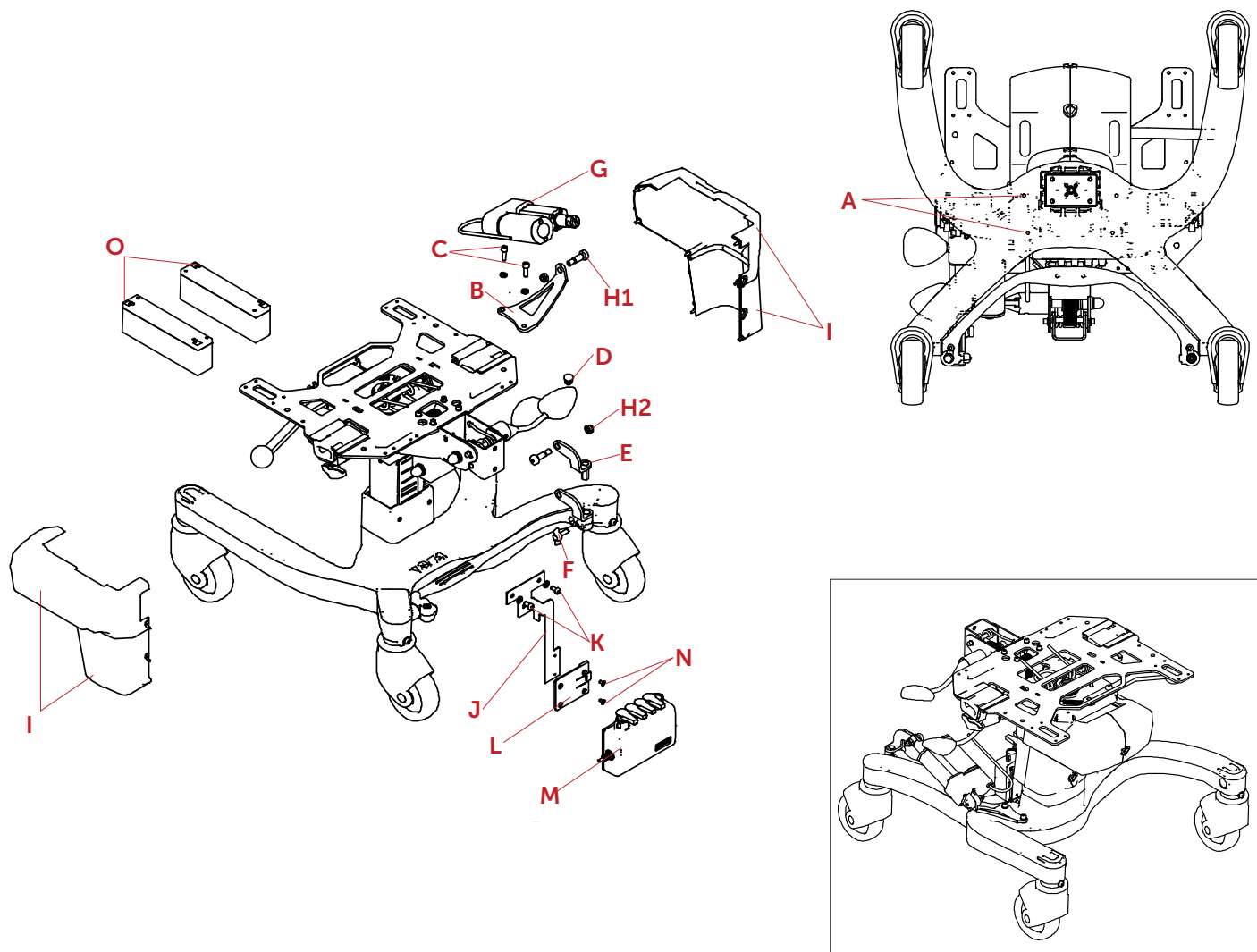
El. bremsekit / El. brake kit / Bremsekit Elektrisk - VELA Tango 700

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El. bremsekit / El. brake kit / Bremsekit

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ENGLISH

The brake should be on the right side of the chair.

1. Drill two Ø5 holes from the bottom of the frame to fixate the fittings. Spots to drill in the frame (A) are already marked. Cut M6 threads.
2. Fixate brackets (B) with 2 x screws and washers (C) in the new threads (A).
3. Remove existing plug (D) and replace with bracket (E). Fasten with wing screw (F).
4. Fit the actuator (G) between the two fittings (B+E). The motor must face the chair. Tighten with shoulder screw and nut (H) at each end.
5. Remove the plastic covers (I) by taking off the o-rings and pulling to the side (see instruction 106231). If necessary use a small screwdriver.
6. Install the brackets for motor control (J) with the same screws (K).
7. Mount the plastic bracket (L) from the motor controller (M) onto the bracket (J) with the screws (N).
8. Slide the motor controller (M) over the plastic bracket (L), push until it clicks into place.
9. Connect the motor control unit to the front on both batteries (O). **Important:** +/- poles must be correctly oriented. The motor control is damaged if it is installed inverted.
10. Connect the actuator (G) and the handbox.
11. Reinstall the covers (I) with the o-rings (see instruction 106231).