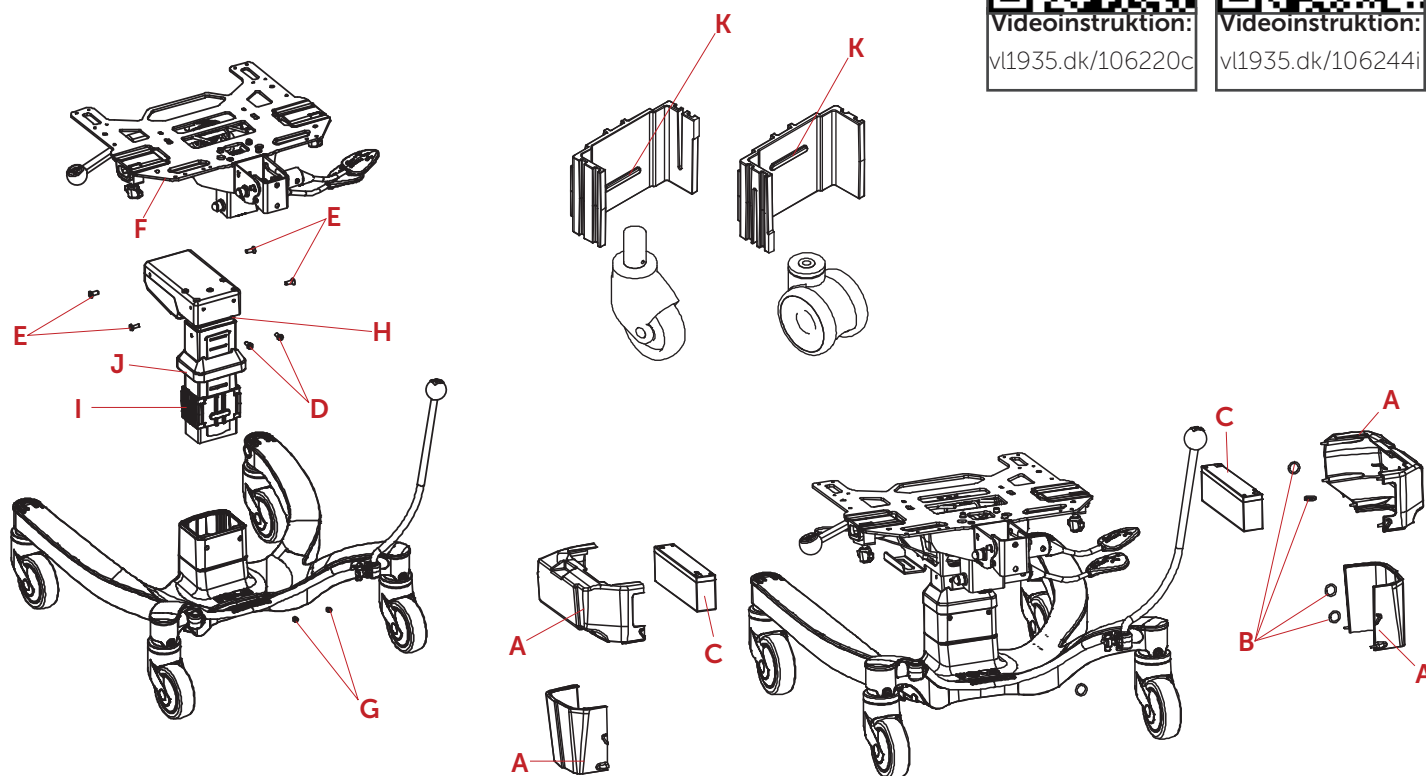
# Aktuator / Actuator / Motor Seteløft - VELA Tango 600ES & 700E

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#### ENGLISH

**Warning: Before use, the actuator MUST be reset! See item 11**

Interval adjustment is done by extending the actuator all the way and follow the points 5-10.

1. Remove the covers (A) by removing o-rings (B) and pull them out to the side. If necessary use a small screwdriver.
2. Disconnect cables from battery terminals and remove batteries (C). Be careful not to short-circuit the battery terminals against the metal parts of the chair.
3. Unscrew the two cylinder head screws (D) from the back using a 4 mm hex key.
4. Unscrew the four countersunk screws (E) with the hex key.
5. The upper part of the chair (F) can now be lifted off.
6. Loosen the two M8 pin screws (G) with a 4 mm hex key. The actuator (H) can now be lifted off the chassis and removed.

7. Remove the plastic bushing (I). Note which track the bush (I) is in, to mount new actuator (H) at the same height.

8. Cover (J) is removed and moved onto the new actuator (H).
9. Replace the plastic bushing (I) on the actuator (H). Remember to turn it correctly depending on the wheel type of the chair.
10. Assemble the rest by doing the above in reverse order. The two M8 pin screws (D) at the back of the column must be tightened with 7 Nm. **Important:** +/- poles must be correctly oriented. The motor control is damaged if it is installed inverted.

**Warning: Before use, the actuator MUST be reset!**

11. Before use, the actuator must be reset to ensure proper operation: Press and hold the up and down buttons (at the same time) for at least 20 seconds until the lift is in the lowest position.

**Note: (K) Lower track on single wheels (K), upper track on double wheels.**