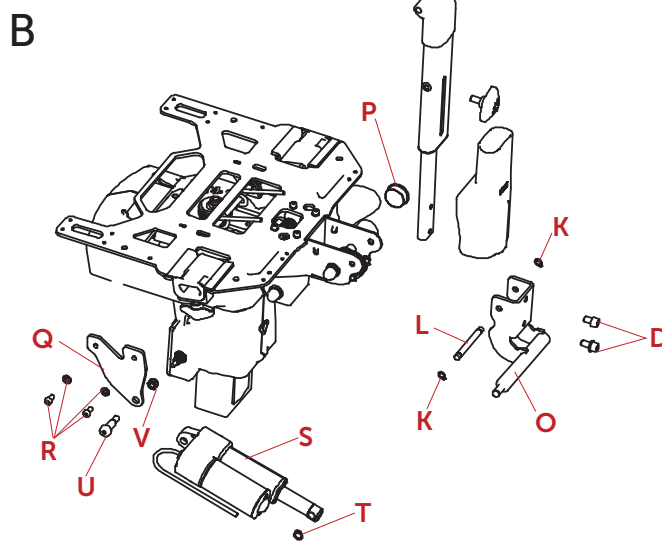
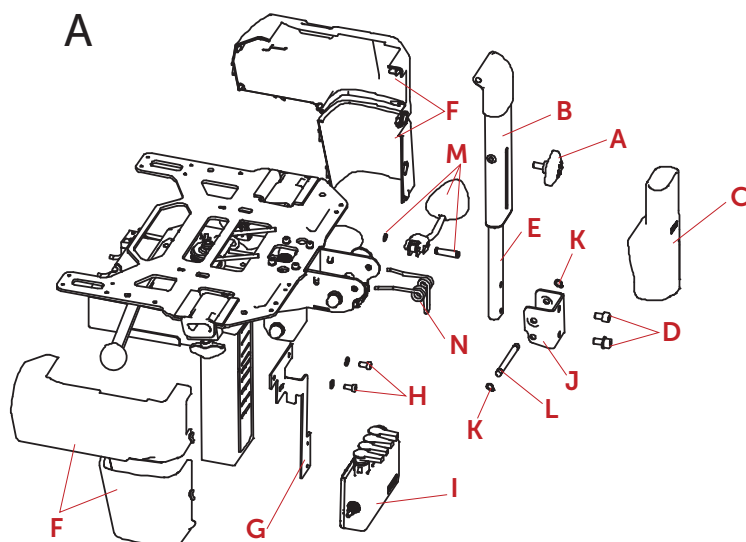
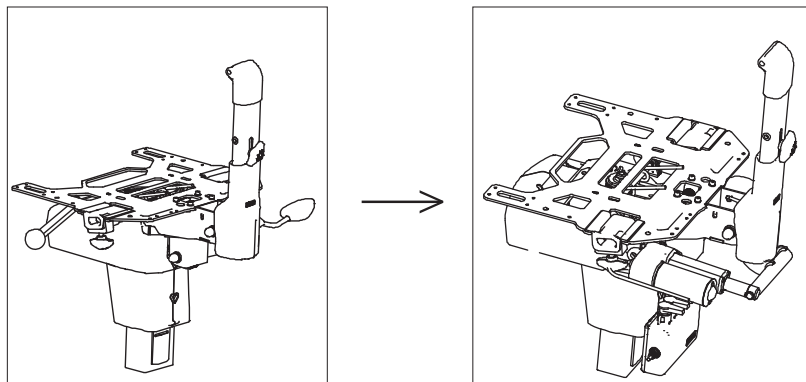
Elryg / El. back / Motor Rygg - VELA Tango 700

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ENGLISH

(A):

1. Remove the backrest from the chair by unscrewing the wing screw (A) completely, lift off the backrest with the back bracket (B) and back cover (C) and set aside.
2. Unscrew the two M8 screws (D) holding the back post (E) and set the back post aside.
3. Remove the covers (F) by removing the o-rings and pulling to the side. Optionally, use a small screwdriver (see instructions 106231). This provides access to the batteries.
4. Mount the bracket for the motor controller (G) with the two screws and washers (H). Rotate the seat if necessary to make more space for mounting the bracket. Rotate the seat back into place and remove the rotation handle (see instructions 106247).
5. Mount the motor controller (I) on the bracket (G) and connect power to the front of the batteries. **Important:** +/- poles must be oriented correctly. If mounted incorrectly, the motor controller will be damaged.
6. Reinstall the covers (F) (see instructions 106231).
7. Remove the backrest console (J) by removing the retaining ring (K) on one side of the axle (L) and pushing it out of the console. The small axle in the backrest console is also pushed out by removing the two retaining rings on the inside. Release the back tilt handle (M) and push the slats slightly away to make better space for the new backrest console.

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8. The backrest console (J) and the large spring (N) are no longer needed. The axle (L) and retaining rings (K) are reused.

9. Remove the handle (M) by removing the small retaining ring and pushing out the axle - these parts are no longer needed.

(B):

10. Install the new electric backrest console (O) and reinsert the axle (L) and retaining rings (K).
11. Place the plastic cap (P) on the tube where the back handle (M) was removed.
12. Mount the bracket (Q) on the left side of the bearing plate with the two screws and washers (R).
13. The actuator (S) is first mounted on the backrest console (J) and the retaining ring (T) is placed on it.
14. The actuator (S) is then tilted up to the bracket (Q) and screwed in place as shown with a pivot bolt (U) and nut (V). The nut should not be fully tightened.
15. Mount the back post (E) on the new backrest console (J) with the two screws (D) and place the back cover (C) on.
16. Secure the back bracket (B) with the wing screw (A).
17. Connect the wires between the actuator, motor controller, and control.