Elektrisk fremdrift / Fremdriftshjelp / Power Wheels - VELA Tango 700E

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Steg 1 - Mekanisk installation under stolen

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Elektrisk fremdrift / Fremdriftshjelp / Power Wheels - VELA Tango 700E

Mounting instruction - Power Wheels (Old versions & Uni2 frame)

Important: BEFORE installing Power Wheels.

- Cannot be mounted on chairs with the lowest seating height. Refer to instruction number **106244** to adjust the chair's height range to a higher seating height.
- Cannot be mounted on chairs with seat rotation. Refer to instruction number **106247** for removing the rotation handle and locking the seat rotation.

ENGLISH

Mounting includes modification of structural components and electrical system. This should only be carried out according to the following instructions and performed by qualified personnel. If not familiar or comfortable with handling this, please contact your VELA supplier or technician.

Prerequisites – tools needed: 4 mm Allen key, 5 mm Allen key, Phillips screwdriver, 6 mm drill bit (not applicable to the Uni2 frame).

Step 1 – Mechanical installation under the chair

1. Brake and lay down the chair on the rear side so you can access underneath the frame.
2. Drill two Ø6 mm holes completely through at the indicated locations. On the Uni2 frame, knock out the two knock-out tabs instead.
3. Mount the boom using the 5 mm Allen key to fix M6 screws from the top side. Note the cables and make sure they are not obstructed.
4. Raise the chair back up again and engage the wheels by pushing them to the floor.

Step 2 – Electrical installation

5. Optional: increase the lift to maximum height adjustment to allow for better working space.
6. Remove the four covers by removing the six o-rings.
7. Disconnect the power on the front of the batteries.
8. Connect the same wires to the piggyback connectors on the drive system. Connect positive to positive and negative to negative. Note the correct sleeving of the cables around the seat mechanism.
9. Connect the piggyback connectors to the batteries. Positive to positive, negative to negative.
10. Use one of the tie wraps to fix the power wire to the back of the seat mechanism together with the controller wire.
11. Install the top covers and three o-rings again. Do not install the bottom two covers yet.

Step 3 – Height limitation installation

12. Optional: increase the lift to maximum height adjustment to allow for better working space.
13. Mount the magnets and metal plate on the right-

hand cover as shown. The component will fit together nicely when orientated correctly.

14. Screw the two plastic brackets on the top of the column outer tube as shown:
15. Assemble the connectors between the height limit sensor and drive controller.
16. Reinstall the last two covers again using three o-rings.

Step 4 – Joystick installation

17. Unscrew the armrest at the height adjustment. The joystick can be mounted on either the right or the left side of the chair. Note it can also be mounted on the inside of the armrest.
18. Unscrew the armrest cushion by removing the two screws using a 4 mm Allen key.
19. Sandwich the joystick under the armrest cushion and armrest bracket. Note that the joystick can be mount in two different ways depending on user preference. Recommendation is as shown to give better space to the user and the fold-away option.
20. Screw in the two screws again using the same tools.
21. Fasten the armrest to the chair again
22. Adjust the joystick controller in the desired depth using a 4 mm Allen key.
23. Connect the joystick controller to the wire which was fastened previously to the back of the seat mechanism.
24. As a last step, fix the cables with tie wraps the indicated positions:

Step 5 – Finishing tests

25. Turn on the drive system.
26. When the lift is at maximum position, the display should read "C1" and not able to drive.
27. When the lift is gradually lowered, the "C1" message will disappear, and the chair is ready to drive.
28. Test that the direction of the joystick movements matches the direction of the intended chair movement.
29. Turn of the drive system again.
30. Your product is now ready for use.