


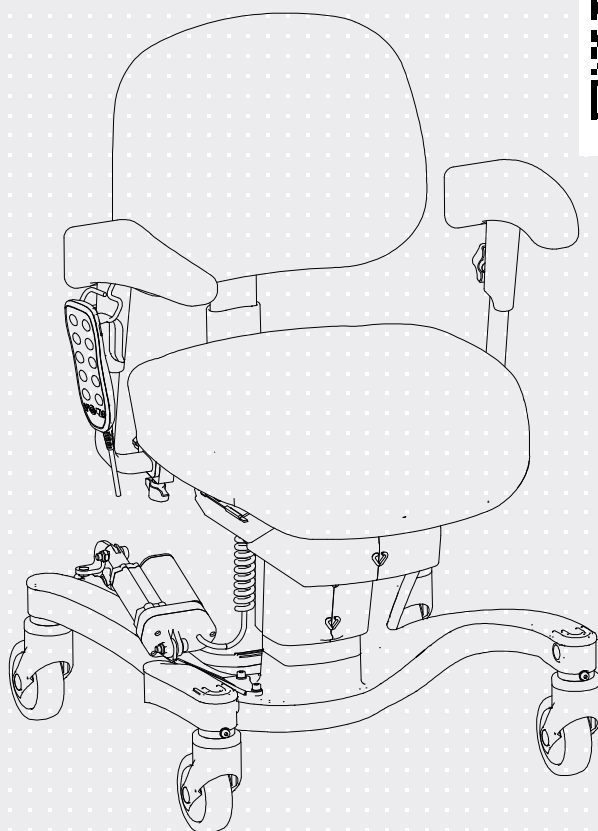

Sit-to-stand

VELA TANGO 700E

Manual. no. 105792

vela.my/trpISO 9001
ISO 14001

HAND CONTROL

How to operate your VELA Tango 700E with sit-to-stand function



SEAT HEIGHT

Adjust your seat height up or down using the arrows. Always set the seat height to match the activity you are performing.

SEAT TILT ADJUSTMENT

The entire seating unit (seat and backrest) moves together. Adjust the seat tilt forward or backward using the arrows until you feel secure in your activity.

BRAKE

The electric brake is engaged and released using the buttons. The chair always applies the brake when you stand up or sit down using the sit-to-stand function.

BACKREST ANGLE ADJUSTMENT

Adjust the backrest forward or backward using the arrows. Set the backrest to a position where you feel comfortable.

SIT-TO-STAND FUNCTION

The brake locks, and the seat tilt, backrest angle, and seat height adjust automatically. Pressing the up arrow supports you in standing up safely, while pressing the down arrow assists in sitting down securely.*

The sit-to-stand function can be customised to provide optimal support for standing up and a neutral sitting position (see guide on the next page).

*This sit-to-stand function has memory, allowing the user to pause the sequence for up to one minute while adjusting their sitting position, and then resume the sequence. The sequence will be canceled if paused for more than a minute or if other buttons are pressed. If canceled, the chair will move into neutral position before a new sequence can begin.

CUSTOMIZE SETTING

Note: If needed, get guidance from a therapist to set an optimal seating position and sit-to-stand position.



STEP 1: CUSTOMIZE YOUR NEUTRAL SEATING POSITION

Make sure the chair is locked! Adjust the seat height, seat tilt, and backrest angle so that you are sitting comfortably and stably with your feet flat on the floor. Once the chair is adjusted, press the two buttons at the bottom right for five seconds. When you hear a click, the setting is saved.

STEP 2: CUSTOMIZE YOUR SIT-TO-STAND POSITION

Make sure the chair is locked! Adjust the seat height, seat tilt, and backrest angle so that you are standing comfortably and stably with your feet flat on the floor. Once the chair is adjusted, press the two buttons at the bottom left for five seconds. When you hear a click, the setting is saved.



Perform a reset of the actuator. Hold both height adjustment buttons (up and down) simultaneously for 20 seconds. Keep holding the buttons until the actuator has fully retracted and then moved approximately 5 mm up again.

CHAIR FUNCTIONS



Backrest height



Backrest angle



Seat depth



Brake



Tilt



Armrest width



Armrest height



Electric height adjustment



Electric sit-to-stand

