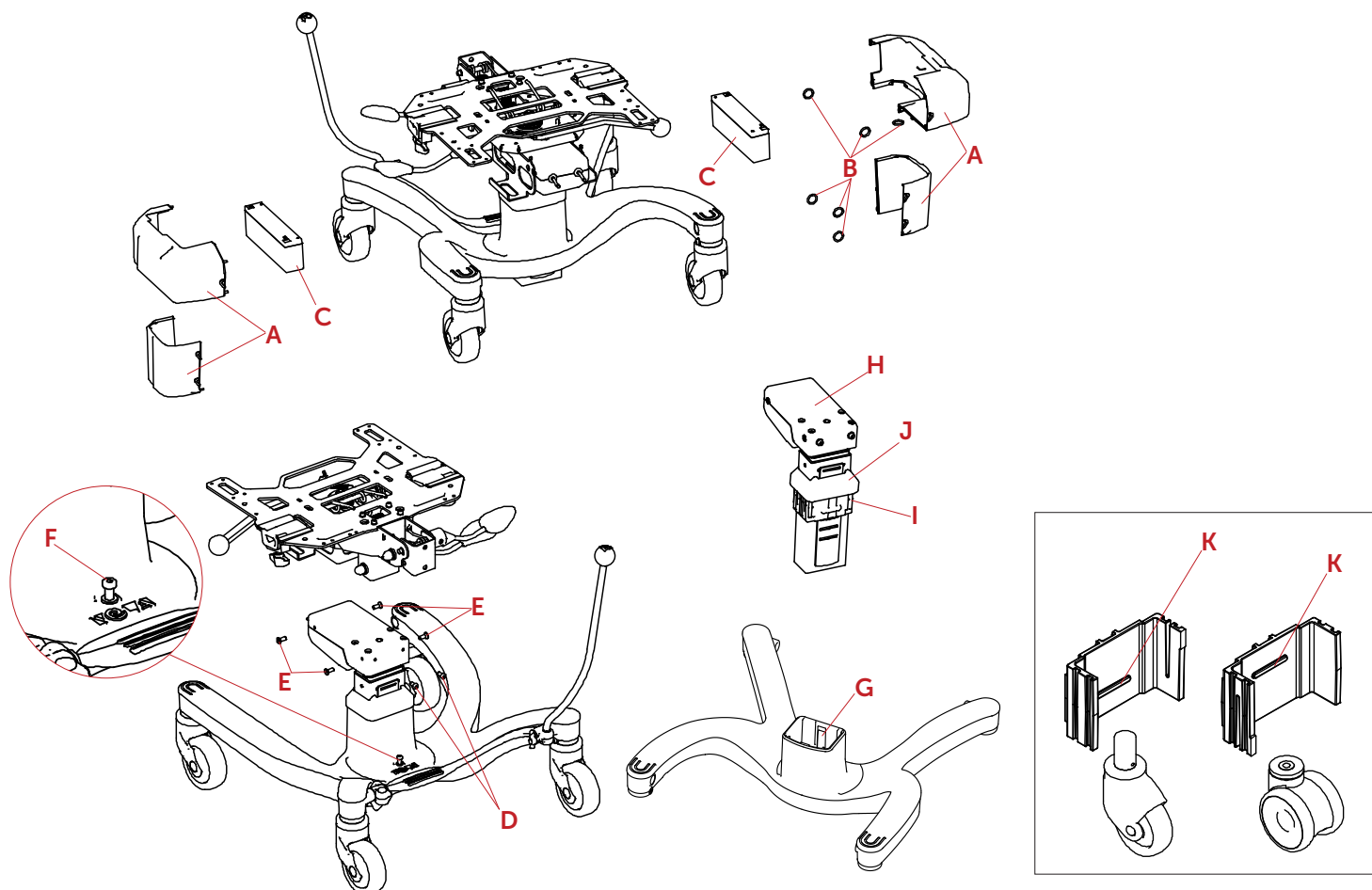
Aktuator / Actuator / Motor Seteløft - VELA Tango 600ES & 700E, Uni2

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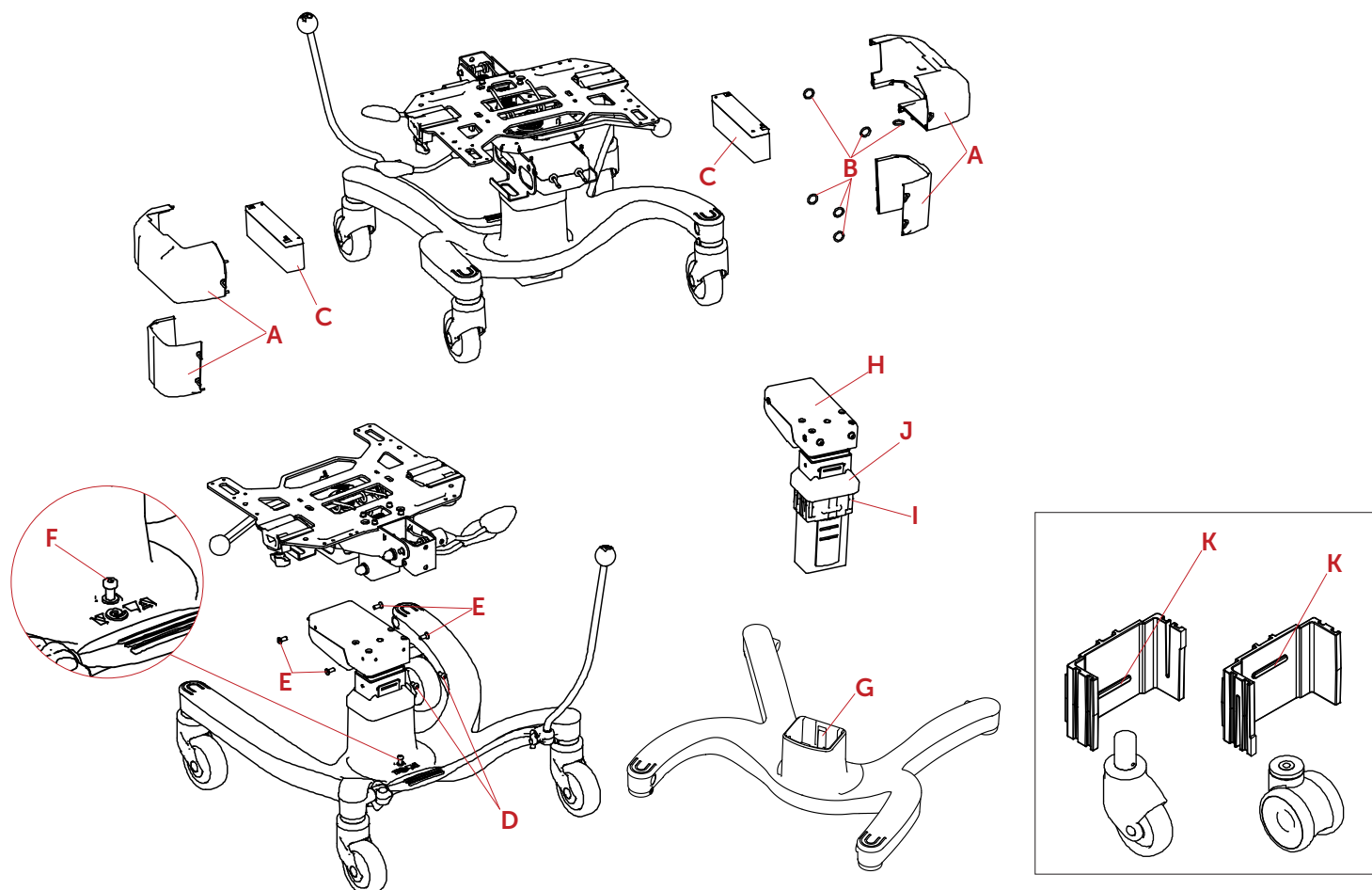
Aktuator / Actuator / Motor Seteløft - VELA Tango 600ES & 700E, Uni2

Mounting Instructions – Replacing the Actuator on the VELA Tango 600ES & 700E with Uni2 Frame

ENGLISH

Warning: Before use, the actuator MUST be reset! See section 13.

1. Remove the covers (A) by removing the O-rings (B) and pulling the covers aside. If necessary, use a small flat-head screwdriver.
2. Disconnect the cables from the battery terminals and remove the batteries (C). Make sure the battery terminals do not short-circuit against any metal parts of the chair.
3. Remove the two socket head cap screws (D) and the four countersunk screws (E) using a 4 mm Allen key.
4. The upper part of the chair can now be lifted off.
5. Loosen screw (F) using a 4 mm Allen key. The screw only needs to be loosened by 10–12 mm. Tap the head of screw (F) with a rubber mallet to loosen the wedge (G) in the frame.
6. The actuator (H) can now be lifted out of the base frame.
7. Remove the plastic bushing (I). Note which groove the rib on the bushing is positioned in so the new actuator can be installed at the same height.
8. Remove cover (J) and transfer it to the new actuator.

9. Install the plastic bushing (I) on the new actuator. Note: Make sure the bushing (I) is oriented correctly according to the chair's wheel type. Rib (K) at the bottom for single-wheel casters; rib (K) at the top for dual-wheel casters.
10. Lower the actuator (H) back into the frame. Make sure the wedge (G) has been pushed back into position. If necessary, press down on screw (F) while lowering the actuator into the frame.
11. Tighten screw (F) securely.
12. Reinstall the upper part of the chair by performing steps 3–1 in reverse order.
Important: The positive (+) and negative (–) battery terminals must be connected correctly. Reversing the polarity will damage the motor controller.

Warning: Before use, the actuator MUST be reset!

13. To ensure proper operation, the actuator must be reset: Press and hold both the Up and Down buttons simultaneously for at least 20 seconds until the lift reaches its lowest position.

Note: Rib (K) at the bottom for single-wheel casters; rib (K) at the top for dual-wheel casters.