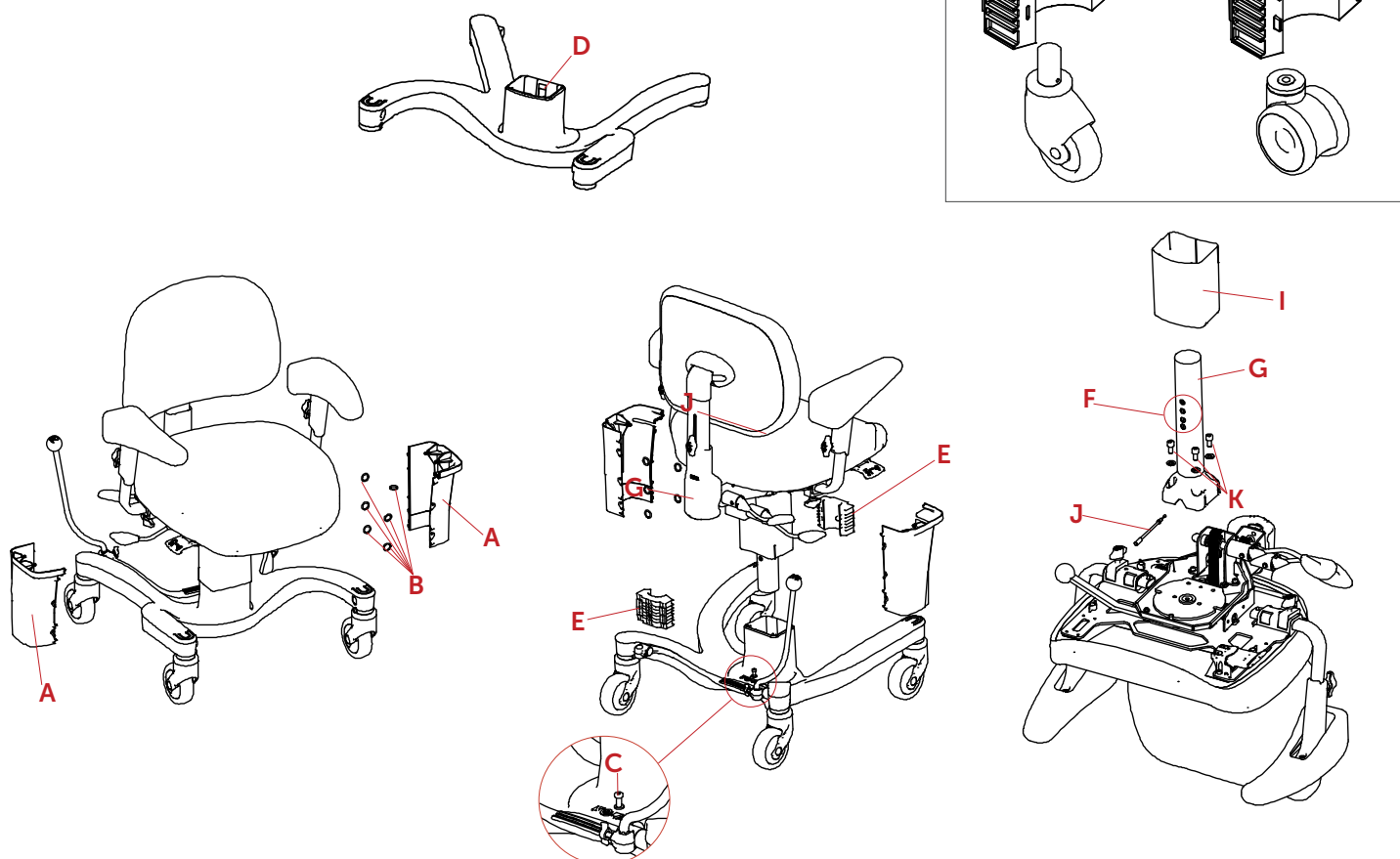
Gasfjeder / Gas spring / Gassfjær - VELA Tango 600S/700, Uni2

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ENGLISH

1. Remove the covers (A) by removing the O-rings (B) and pulling the covers aside. If necessary, use a small flat-head screwdriver or a ballpoint pen to remove the O-rings.
2. Loosen screw (C) using a 4 mm Allen key. The screw only needs to be loosened by 10–12 mm.
3. Tap the head of screw (C) with a rubber mallet to loosen the wedge (D) in the frame.
4. Lift the upper part of the chair off the base frame and remove the two plastic bushings (E).
5. Remove cover (I) from the gas spring (G).
6. Remove the gas spring (G), including the bracket and cable (J), by removing the four screws (K) with a 6 mm Allen key.
7. Pull out the cable (J)—use needle-nose pliers if necessary — and install the cable (J) in the new gas spring (G).
8. Secure the new gas spring (G) with the four screws (K).
9. Install cover (I) on the gas spring.
10. Reinstall the two plastic bushings (E) in the correct hole (F). Make sure they are oriented correctly according to the chair's wheel type.
11. Lower the upper part of the chair with the new gas spring back into the frame. Make sure the wedge (D) has been pushed back into position. If necessary, press down on screw (C) while lowering the gas spring into the frame. Tighten screw (C).
12. Slide cover (I) down toward the frame.
13. Secure the covers (A) beneath the seat using the O-rings (B).

Note: Pin (H) faces downward for single-wheel casters and upward for dual-wheel casters.