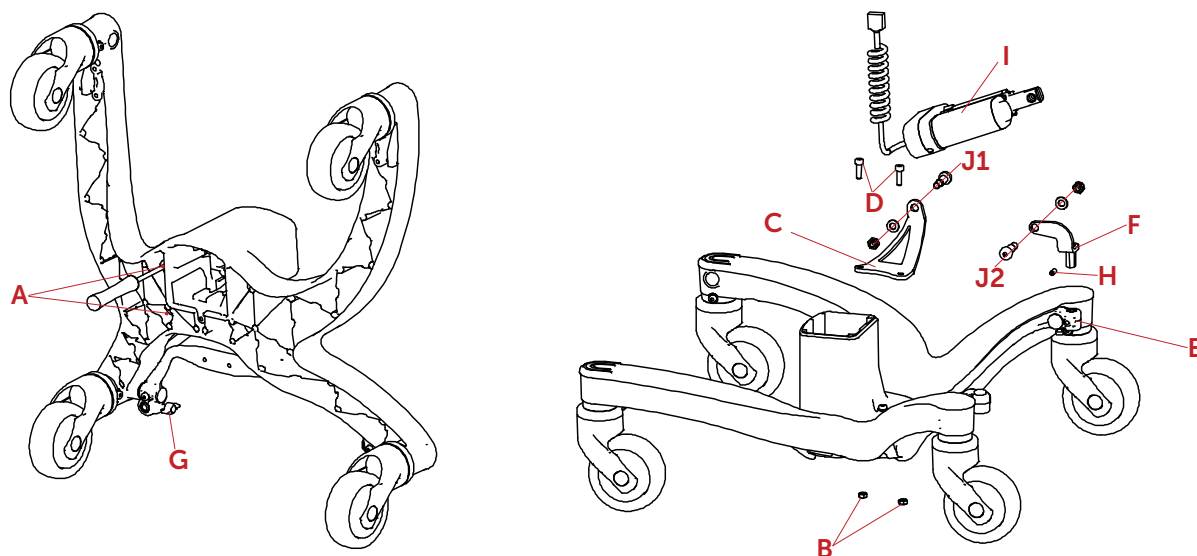
El. bremsekit / Electric brake kit / Bremsekit Elektrisk - VELA Tango 600ES/700E, Uni2

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ENGLISH

The brake must be installed on the right side of the chair:

1. Lay the chair down and knock out the two knock-out tabs (A) from underneath using a screwdriver. Press the two nuts (B) into the holes. **Note:** When combined with the Power Wheels, the nuts must not be used.
2. Secure bracket (C) with the two screws (D) in the new holes using the installed nuts (B).
3. Remove the brake rod from the brake bracket (E) and install bracket (F) instead. (Optional) Replace the wing screw (G) with a set screw (H), if required.
4. Position the actuator (I) between the two installed brackets (C+F). The wide side of the actuator must face toward the chair.
5. Secure the actuator (I) with the clevis pin, washer, and nut (J1 & J2) at both ends. **Important:** The clevis pin and nut must be installed correctly in the actuator piston rod.

Connecting the electronics:

6. Remove the seat rotation (see instruction 106247).
7. Remove the plastic covers and O-rings (see instruction 106231).
8. Mount bracket (K) for the motor controller on the support plate (L) using the screws (M).
9. Slide the motor controller (N) onto bracket (K).
10. Connect the motor controller to the power supply at the front of both batteries (O). **Important:** Make sure positive is connected to positive and negative to negative.
11. Make sure the cable routing does not cause cables to become pinched or stretched, for example when using the chair's seat tilt function.
12. Connect the cables to both the actuator and the control unit.
13. Reinstall the covers with the O-rings (see instruction 106231).